





Instructions for Use:

- 1 Place the patient on top of the Complete Cair Overlay, aligning the top edge with the patient's shoulders.
- 2 Reaching over the patient, pull the two red side handles to turn the patient toward you, allowing a second caregiver to place a Flo-Lock Positioner beneath the black nylon flap.
- 3 Gently lower the patient into the Flo-Lock Positioner, allowing it to self-mold to the patient's body. Tuck the black nylon flap beneath the Flo-Lock positioner to secure it in place.
- 4 To clean the product, follow standard hospital protocol.

Key Features:

- **Side handles** allow for safe patient handling during transfers and repositioning
- **Flo-Lock Positioner** uses self-molding technology to maintain therapeutic positioning
- **Non-powered** design eliminates the need for a manual or electric air pump
- **Nylon Flap** secures the positioner in place, preventing patient migration / sliding

Additional Considerations:

- Recommend for single patient use
- Underpads and linens can be utilized as per hospital protocol
- DEHP and latex free
- Limited 90-day warranty against manufacturer defects

