

### Indication for Use:


Spry's Flo-Form™ Positioner easily contours to the patient's body and maintains its shape until remolded. In doing so, Flo-Form™ redistributes and offloads bony prominences in key areas for improved patient outcomes.

### Product Features:



- **Flo-Form™** is a non-powered fluidized medium, designed to maximize contouring and support
- **The Flo-Form™ Positioner** will not flow on its own, but it effectively contours when normal body forces are applied (regardless of the direction of the force)
- **Redistributes pressure** over a greater surface area
- **Effectively positions patients** and promotes neutral spine and limb alignment

### Instructions for Use:

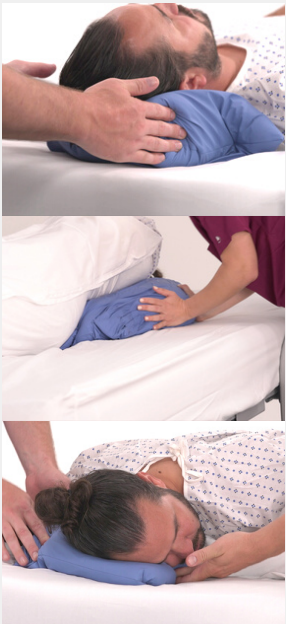

- 1 Place positioner under patient. Be sure to place a barrier, such as a cover or sheet, between the positioner and patient's skin.
- 2 Mold positioner to patient by pressing from the perimeter toward the patient.
- 3 Positioner can be molded to accommodate most medical equipment.
- 4 Remold as needed for multiple position changes.
- 5 Do not use if positioner has been torn or punctured.

### Additional Considerations:

- Clean topically using hospital-approved products
- DEHP and latex free
- Limited 90-day warranty against manufacturer defects

