

## Indications for Use:

For patients requiring special positioning to accommodate individual clinical needs, as per facility protocols.

## Product Features:



- **Flo-Lock self-molding technology** contours to redistribute pressure over a larger area
- **Pneumapure filter technology** and waterproof exterior allow positioner to breathe while blocking pathogen ingress and egress
- **Lightweight**, yet durable design supports up to 350 lbs
- **Compatible** with all non-powered turning and repositioning systems

## Instructions for Use:



- 1 Place positioner under patient. Be sure to place a barrier, such as a cover or sheet, between the positioner and patient's skin.
- 2 Positioner will naturally adjust and contour to support the patient's body.
- 3 Positioner can be adjusted to accommodate most medical equipment
- 4 Do not use if positioner has been torn or punctured.

## Additional Considerations:

- Clean topically using hospital-approved products
- DEHP and latex free
- Limited 90-day warranty against manufacturer defects

