


Indications for Use:

To support the head, or other body part, during sleep or rest.

Product Features:

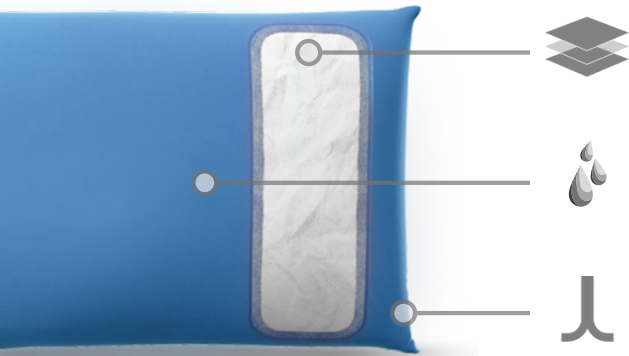

- Patented **Pneumapure® filter technology** blocks up to .02 micron (including viruses, bacteria, and other pathogens), while maintaining airflow
- **Waterproof exterior** prevents pathogen ingress and makes pillow easy-to-clean with no laundry needed
- **Hermetically-sealed** seams keeps pathogens out by eliminating stitching holes

General Guidance

- A breathable pillowcase is recommended for patient comfort and safety
- Inspect the product prior to use:
 - No holes or damage appear on the product
 - All pillow casing seals are closed and no tears or holes are present
- The Spry Pure PPE Pillow cannot be repaired. If damaged (punctured, torn, seals opened, etc.), it should be replaced immediately.

Cleaning Instructions

- Product should be cleaned topically, using hospital approved products

Disposal

- No hazardous material present
- Contaminated products should be disposed of according to hospital policy and local environmental regulation

Additional Considerations:

- Do not place the product on or near a heat source
- Do not allow sharp objects to penetrate the exterior of the product
- Not recommended for newborns or infants