

Indication for Use:

To aid in providing developmental positioning for neonatal patients while redistributing pressure on sensitive skin.

Product Features:



- **Flo-Form** is a non-powered fluidized medium, designed to maximize contouring and support
- **The Flo-Form Positioner** will not flow on its own, but it effectively contours when normal body forces are applied (regardless of the direction of the force)
- **Redistributes pressure** over a greater surface area
- **Effectively positions patients** and promotes neutral spine and limb alignment

Instructions for Use:

- 1 Take fluidized positioner out of plastic bag and pre-warm in a blanket warmer (be sure not to exceed 36° Celsius).
- 2 Place the pre-warmed fluidized positioner under the the specific body section you are trying to position. Be sure to place a barrier, such as a cover or sheet, between the positioner and the baby's skin.
- 3 Gently stabilize the baby on the positioner with one hand, then slowly push the perimeter of the positioner towards the baby.
- 4 Remold as needed for medical equipment and position changes.

Additional Considerations:

- Recommended for single patient use
- Clean topically with hospital-approved products
- Do not use if positioner has been torn or punctured
- DEHP and latex free
- Limited 90-day warranty against manufacturer defects

