

Instructions for Use:



- 1 Place the patient on top of the Complete Cair Overlay, aligning the top edge with the patient's shoulders.



- 2 Reaching over the patient, pull the two red side handles to turn the patient toward you, allowing a second caregiver to place a Flo-Lock Positioner beneath the black nylon flap.



- 3 Gently lower the patient into the Flo-Lock Positioner, allowing it to self-mold to the patient's body. Tuck the black nylon flap beneath the Flo-Lock positioner to secure it in place.



- 4 To clean the product, follow standard hospital protocol.

Key Features:

- **Side handles** allow for safe patient handling during transfers and repositioning
- **Flo-Lock Positioner** uses self-molding technology to maintain therapeutic positioning
- **Non-powered** design eliminates the need for a manual or electric air pump
- **Nylon Flap** secures the positioner in place, preventing patient migration / sliding

Additional Considerations:

- Recommend for single patient use
- Underpads and linens can be utilized as per hospital protocol
- DEHP and latex free
- Limited 90-day warranty against manufacturer defects



Cleaning Instructions:

- The Complete Cair Turning & Repositioning System should be disinfected by wiping the casing with a hospital-approved disinfectant. (Suggested max 72% alcohol, max 1000ppm active chlorine.) Iodine and blood stains can be removed with a hospital-approved stain remover.
- After the disinfection, all products should be wiped down using a clean cloth moistened with clean water. Allow the to dry fully before use.
- Bleaching, scrubbing, machine washing, tumble drying, dry cleaning, autoclaving, or ironing are not recommended.
- The Pneumapure filters are welded onto the positioners and do not require changing. The filter should last as long as the rest of the product, as long as it is properly maintained (not punctured or torn). The filter porosity is at the micro-level. It allows air to transfer, but fluids/solids/germs will not penetrate into the interior of the product.

If you have a specific protocol that our suggested cleaning doesn't fall under, please let us know, and we can determine if it will negatively affect the life of the product. The product should be cleaned as often as you would clean your standard hospital pillows.