

Indication for Use:

To support the body during sleep or rest

Product Features:




- **Patented Pneumapure filter technology** allows airflow while blocking pathogens such as MRSA, C. diff, E. coli, coronavirus, and more
- **Waterproof exterior** prevents pathogen ingress and makes cushion easy-to-clean with no laundry needed
- **Hermetically sealed** seams keep pathogens out by eliminating stitching holes

General Guidance:

- The Spry Pure neonatal mattress is designed to support the body during sleep or rest. A breathable sheet is recommended.
- Inspect the mattress before use to ensure that:
 - No holes or damage appear.
 - The Pneumapure® Filter is firmly attached to the mattress casing.
 - All mattress casing seals are closed, and no tears or holes are present.
- The use of developmental care products, such as Flo-Form positioners is suggested with any Spry Pure NICU mattress.
- The Spry Pure mattress cannot be repaired. If damaged (punctured, torn, seals opened, etc.), it should be replaced immediately.

Additional Considerations:

- Suitable for multi-patient use
- Clean topically with hospital-approved products
- DEHP and latex free
- Limited 90-day warranty against manufacturer defects

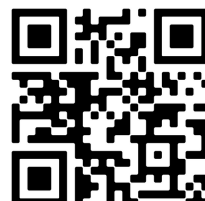

Cleaning Instructions:

- The Spry Pure neonatal mattress should be disinfected by wiping the casing with a hospital-approved disinfectant. (Suggested max 72% alcohol, max 1000ppm active chlorine.) Iodine and blood stains can be removed with a hospital-approved stain remover.
- After the disinfection, the mattress should be wiped down using a clean cloth moistened with clean water. Allow it to dry fully before use.

If you have a specific protocol that our suggested cleaning doesn't fall under, please let us know, and we can determine if it will negatively affect the life of the product. The product should be cleaned as needed during use.