

## Indications for Use:

Spry's Static Air Seat Cushion is used to redistribute skin pressure for the prevention and treatment of pressure ulcers.

## Product Features:



- **Static Air technology** uses air displacement to relieve and equalize pressure
- **Single air chamber** design allows for low-profile use
- **High-quality, medical-grade vinyl** is durable and easy to clean

## Instructions for Use:

Use as directed by the appropriate clinician. Always follow your facility protocol for patient seating.

- 1 Take out of the package and unfold.
- 2 Roll cushion from one side. If properly inflated, it should roll halfway before resisting.
- 3 Unroll the cushion and place it on the chair.
- 4 Place the patient with the buttocks centered on cushion.



Our Static Air technology uses air displacement and a single low-profile air chamber that is designed for maximum contouring and pressure redistribution. The low-profile cushion needs a very small column of air to yield proper comfort and pressure relief. A properly inflated cushion can be easily rolled to the halfway point and will and should appear almost flat until you sit on it.

**Sizing:** standard 20 x 20, bariatric 24 x 31

**Weight Limit:** standard 350 lbs, bariatric 600 lbs

- Suitable for multi-patient use
- DEHP and latex free
- Limited 30-day warranty against manufacture defects



## Flo-Form- Neo/Peds/Adult

- The Spry Flo-Form positioner should be disinfected by wiping the casing with a hospital-approved disinfectant. (Suggested max 72% alcohol, max 1000ppm active chlorine.) Iodine and blood stains can be removed with a hospital-approved stain remover.
- After the disinfection, the positioner should be wiped down using a clean cloth moistened with clean water. Allow the pillow to dry fully before use.
- Bleaching, scrubbing, machine washing, tumble drying, dry cleaning, autoclaving, or ironing are not recommended.

## Pnuemapure Neonatal Mattress

- The mattress should be disinfected by wiping the casing with a hospital-approved disinfectant. (Suggested max 72% alcohol, max 1000ppm active chlorine.) . Iodine and blood stains can be removed with a hospital-approved stain remover.
- After the disinfection, the mattress should be wiped down using a clean cloth moistened with clean water. Allow to dry fully before use.
- Bleaching, scrubbing, machine washing, tumble drying, dry cleaning, autoclaving, or ironing are not recommended.
- The Pneumapure filter is welded onto the mattress and does not require changing. The filter should last as long as the rest of the product, as long as it is properly maintained (not punctured or torn). The filter porosity is at the micro-level. It allows air to transfer, but fluids/solids/germs will not penetrate into the interior of the product.

## Seat Cushion

- The Spry seat cushion should be disinfected by wiping the casing with a hospital-approved disinfectant. (Suggested max 72% alcohol, max 1000ppm active chlorine.) Iodine and blood stains can be removed with a hospital-approved stain remover.
- After the disinfection, the product should be wiped down using a clean cloth moistened with clean water. Allow the pillow to dry fully before use.
- Bleaching, scrubbing, or autoclaving are not recommended.