

Indication for Use:

Provides positioning for neonatal, pediatric, and adult patients while redistributing pressure on sensitive skin. Flo-Form™ offloads bony prominences in key areas for improved patient outcomes.

Product Features:



- **Flo-Form** is a non-powered fluidized medium, designed to maximize contouring and support
- **The Flo-Form Positioner** will not flow on its own, but it effectively contours when normal body forces are applied (regardless of the direction of the force)
- **Redistributes pressure** over a greater surface area
- **Effectively positions patients** and promotes neutral spine and limb alignment

Instructions for Use:

- 1 Take the positioner out of the plastic bag & place it under the patient. Be sure to place a barrier, such as a cover or sheet, between the positioner and the patient's skin.
- 2 Place the fluidized positioner under the specific body section you are trying to position. Mold positioner by pressing from the perimeter toward the patient.
- 3 Remold as needed to accommodate medical equipment and position changes.

Additional Considerations:

- Do not use if positioner has been torn or punctured
- DEHP and latex free
- Limited 90-day warranty against manufacturer defects



SCAN ME