



Cleaning Instructions

Flo-Form- Neo/Peds/Adult

- The Spry Flo-Form positioner should be disinfected by wiping the casing with a hospital-approved disinfectant. Iodine and blood stains can be removed with a hospital-approved stain remover.
- After the disinfection, the positioner should (but not required) be wiped down using a clean cloth moistened with clean water. Allow the pillow to dry fully before use.
- Straight bleaching, scrubbing, machine washing, tumble drying, or autoclaving are not recommended. See attached list of approved disinfectants.

If you have a specific protocol that our suggested cleaning doesn't fall under, please let us know, and we can determine if it will negatively affect the life of the product. The product should be cleaned between patients and as needed during use.