



## Cleaning Instructions Flo-Lock Positioner

- The Spry Flo-Lock positioner should be disinfected by wiping the casing with a hospital-approved disinfectant. Iodine and blood stains can be removed with a hospital-approved stain remover.
- After the disinfection, the positioner should be wiped down using a clean cloth moistened with clean water. Dry fully before use.
- Straight bleaching, scrubbing, machine washing, tumble drying, or autoclaving are not recommended. See attached list of approved disinfectants.
- The Pneumapure filter is welded onto the positioner and does not require changing. The filter should last as long as the rest of the product, as long as it is properly maintained (not punctured or torn). The filter porosity is at the micro-level. It allows air to transfer, but fluids/solids/germs will not penetrate into the interior of the product.

If you have a specific protocol that our suggested cleaning doesn't fall under, please let us know, and we can determine if it will negatively affect the life of the product. The product should be cleaned between patients and as needed during use.