

## Indications for Use:

For patients requiring special positioning to accommodate individual clinical needs, as per facility protocols.

## Product Features:



- **Flo-Lock self-molding technology™** contours to redistribute pressure over a larger area
- **Pneumapure® filter technology** and waterproof exterior allow positioner to breathe, while blocking pathogen ingress and egress
- **Lightweight**, yet durable design
- **Compatible** with all non-powered turning and repositioning systems

## Instructions for Use:



- 1 Place positioner under patient, using a barrier, such as a sheet (for lateral turning) or flat pillowcase (for heel floating and calf immersion), between positioner and patient's skin.\*
- 2 Positioner will naturally adjust and contour to support the patient's body.
- 3 Adjust positioner to accommodate any medical equipment
- 4 Do not use if positioner has been torn or punctured.

\*Photos are for demonstration purposes only and therefore do not show the use of covers or sheets.

## Additional Considerations:

- Recommended for single patient use
- DEHP and latex free
- Limited 30-day warranty against manufacturer defects

