

Instructions For Use Neonatal Mattress Pad

Product Codes vary by product

Indication for Use:

To support the body during sleep or rest

Product Features:



- Patented Pneumapure filter technology allows airflow while blocking pathogens such as MRSA, C. diff, E. coli, coronavirus, and more
- Waterproof exterior prevents pathogen ingress and makes cushion easy-to-clean with no laundry needed
- Hermetically sealed seams keep pathogens out by eliminating stitching holes

General Guidance:

- The Spry Pure neonatal mattress is designed to support the body during sleep or rest. A breathable sheet is recommended.
- Inspect the mattress before use to ensure that:
 - No holes or damage appear.
 - The Pneumapure® Filter is firmly attached to the mattress casing.
 - All mattress casing seals are closed, and no tears or holes are present.
- The use of developmental care products, such as Flo-Form positioners is suggested with any Spry Pure NICU mattress.
- The Spry Pure mattress cannot be repaired. If damaged (punctured, torn, seals opened, etc.), it should be replaced immediately.

Additional Considerations:

- Suitable for multi-patient use
- Clean topically with hospital-approved products
- DEHP and latex free
- Limited 90-day warranty against manufacturer defects

