



Head and Neck Positioning



Place flattened positioner under patient's head, from the bottom of the neck to the top of the shoulders



Mold by firmly pushing from the sides inward, filling the gap between the mattress and head/neck



Mold away from the patient's ears, and any lines or tubes as necessary



Mold the positioner to offload the occiput as needed

For Proning



Flatten the positioner and mold a depression in the positioner for the ear



Mold positioner away from the face and any lines or tubes as necessary. Mold the positioner around the back of the head

DO:

- Clean with hospital approved disinfectant
- Mold the product away from the back of the head to offload the occiput
- Use with a pillowcase

DON'T:

- Fold the product around the patient's ears/head
- Use the product if it is punctured or not holding its shape



Head and Neck Positioning



Place flattened positioner under patient's head, from the bottom of the neck to the top of the head



Mold by firmly pushing from the sides inward, filling the gap between mattress and head / neck



Mold away from the patient's ears, and any lines or tubes



Mold away the positioner as-needed to offload the occiput

DO:

- Clean with hospital-approved disinfectant
- Mold the product away from the back of the head to offload the occiput

DON'T:

- Fold the product around the patient's head
- Use a product if it is punctured or damaged