

## Head and Neck Positioning



Place flattened positioner under patient's head, from the bottom of the neck to the top of the shoulders



Mold by firmly pushing from the sides inward, filling the gap between the mattress and head/neck



Mold away from the patient's ears, and any lines or tubes as necessary



Mold the positioner to offload the occiput as needed

## Proning



Flatten the positioner and mold a depression in the positioner for the ear



Mold positioner away from the face and any lines or tubes as necessary. Mold the positioner around the back of the head



### DO:

- Clean with hospital-approved disinfectant
- Mold the product away from the back of the head to offload the occiput

### DON'T:

- Fold the product around the patient's head
- Use a product if it is punctured or damaged