

By leveraging the technologies of multiple Spry products, the Complete Cair Proning System allows caregivers to safely and effectively prone their patient's while simultaneously protecting the skin.



### Instructions for Use:

- 1 In a supine position, place flat sheet on top of patient. Place proning pad on top of patient, aligning the air chamber with the clavicle (clear of any lines / tubes).
- 2 Roll patient to the appropriate side and tuck the black edges and long turning handles under patient.
- 3 Roll patient to opposite side to locate tucked handles (beneath patient). Feed the top handles through the bottom handles to secure the patient.
- 4 Once handles are interlocked, place slip sheet beneath the patient and slide them toward the edge of the bed, using the secured handles.
- 5 Hand caregivers the interlocked handles for safe turning, then transition patient into the prone position. Once patient is safely centered on proning pad, tuck the black edges and handles beneath the patient.
- 6 To help prevent skin breakdown, position the head/neck with Flo-Form Positioner and position shins with Flo-Lock Positioner to offload bony prominences and promote neutral spine alignment.

#### Additional Considerations:

- Recommend for single patient use
- DEHP and latex free
- Limited 90-day warranty against manufacturer defects

