

Flo-Form[®] NICU Positioners: Utility & Tube Combo



1. Begin by flattening the utility positioner.



2. Position baby on utility positioner and press outer edges of the positioner to mold it along the baby's perimeter.



3. Wrap fluidized tube around utility positioner, going under any lines or tubing. Use velcro strap to secure both ends of the tube.



4. Squeeze fluidized tube to flatten and build maximum height around the perimeter.



5. Achieve further customization by molding to the desired position.



6. Secure with tether straps where appropriate.



For Proning:

Build prone roll from hips to top of head by pushing down with fingers along side of body. Compress utility positioner below umbilicus & knees and mold larger utility positioner to the baby.



For Supination:

Mold utility positioner around the outer perimeter of the baby's body, being sure to avoid any lines and tubing.