

Indications for Use:

For patients requiring special positioning to accommodate individual clinical needs, as per facility protocols.

Features + Benefits:

- Flo-Lock self-molding technology contours to redistribute pressure over a larger area
- Pneumapure filter technology and waterproof exterior allow positioner to breathe, while blocking pathogen ingress and egress
- Lightweight, yet durable design
- Compatible with all non-powered turning and repositioning systems



Instructions for Use:

- 1 Place positioner under patient, using a barrier, such as a sheet (for lateral turning) or flat pillowcase (for heel floating and calf immersion), between positioner and patient's skin.
- 2 Positioner will naturally adjust and contour to support the patient's body.
- 3 Adjust positioner to accommodate any medical equipment
- 4 Do not use if positioner has been torn or punctured.

Note:

- Clean with hospital approved disinfectants following current protocol
- Photos are for demonstration purposes only and therefore do not show the use of covers or sheets

