

Indications for Use:

For patients that need to be transferred to a prone position to accommodate individual clinical needs, as per facility protocols.

System Components:



Flo-Form Positioner
(for Head and Neck)



Complete Cair
Proning Pad
(for Torso)



Flo-Lock Positioner
(for Legs)

Instructions for Use:

- 1 In a supine position, place flat sheet on top of patient. Place proning pad on top of patient, aligning the air chamber with the clavicle (clear of any lines / tubes).
- 2 Roll patient to the appropriate side and tuck the black edges and long turning handles under patient.
- 3 Roll patient to opposite side to locate tucked handles (beneath patient). Feed the top handles through the bottom handles to secure the patient.
- 4 Once handles are interlocked, place slip sheet beneath the patient and slide them toward the edge of the bed, using the secured handles.
- 5 Hand caregivers the interlocked handles for safe turning, then transition patient into the prone position. Once patient is safely centered on proning pad, tuck the black edges and handles beneath the patient.
- 6 To help prevent skin breakdown, position the head/neck with Flo-Form Positioner and position shins with Flo-Lock Positioner to offload bony prominences and promote neutral spine alignment.